
AeroChamber Plus Z STAT™

Anti-Static Valved Holding Chamber

MOUTHPIECE
LARGE MASK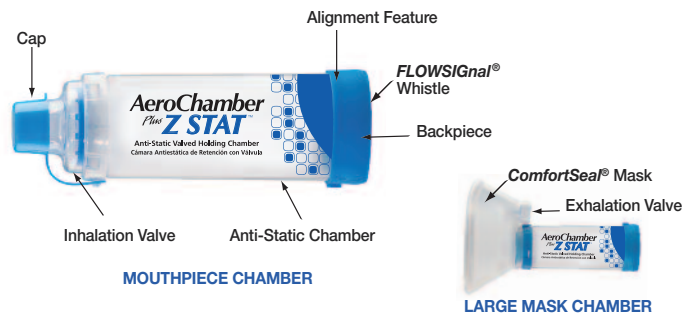

INDICATIONS FOR USE

This product is intended to be used by patients who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers. The intended environments for use include the home, hospitals and clinics.

INSTRUCTIONS FOR USE

THIS PRODUCT CAN BE USED RIGHT OUT OF PACKAGE. BEFORE USE, ENSURE THESE INSTRUCTIONS AND THE INSTRUCTIONS SUPPLIED WITH THE METERED DOSE INHALER (MDI) HAVE BEEN READ.

MOUTHPIECE

LARGE MASK

1		2		3		4		5		6		7	
	Carefully examine the product for damage, missing parts or foreign objects. Remove any foreign objects prior to use. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the Metered Dose Inhaler (MDI) alone until a replacement is obtained. If the patient's symptoms worsen, please seek immediate medical attention.	Remove cap(s) from the MDI and chamber (if applicable).	Shake the MDI immediately before each use as per the instructions supplied with the MDI.	Insert the MDI into the backpiece of the chamber.	Put mouthpiece into mouth and close lips around it to ensure an effective seal.	Apply mask to face and ensure an effective seal.	Breathe out gently and depress the MDI at the beginning of a slow inhalation. Breathe in slowly and deeply through the mouth until a full breath has been taken. Hold breath for 5 – 10 seconds, if possible. Otherwise, keep lips tight on the mouthpiece to maintain seal for 2 – 3 breaths after the MDI is depressed. Slow down inhalation if you hear the FLOWSignal® Whistle sound. Administer one (1) puff at a time.	Breathe out gently and depress the MDI at the beginning of a slow inhalation. Maintain seal for 5 – 6 breaths after the MDI is depressed. Slow down inhalation if you hear the FLOWSignal® Whistle sound. Administer one (1) puff at a time.	Follow instructions supplied with the MDI on how long to wait before repeating steps 3 – 6 as prescribed.				

CLEANING INSTRUCTIONS FOR MASK AND MOUTHPIECE CHAMBERS

THIS PRODUCT CAN BE USED RIGHT OUT OF THE PACKAGE AND THEN CLEANED WEEKLY.

1		2		3		4	
---	--	---	--	---	--	---	--

Notes:

- Storage and operating range 5° C - 40° C (41° F - 104° F) at 15 to 95% relative humidity.
- Product may need to be replaced after 12 months of use. Environmental conditions, storage and proper cleaning can affect product life span.
- THIS PRODUCT CONTAINS NO LATEX.
- Do not share this medical device.
- Clarity of the chamber is a result of the properties of the StatBan® anti-static material.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- Dishwasher validated up to 158°F (70°C).

CAUTIONS:

- PRODUCT MAY BE PERMANENTLY DAMAGED IF BOILED, STERILIZED OR CLEANED IN A DISHWASHER AT TEMPERATURE ABOVE 158°F (70°C).
- Do not leave the chamber unattended with children.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

Product Reorder Number:

MOUTHPIECE: 79710/case 10, 79750/case 50

LARGE MASK: 80710/case 10

Rx Only

Small Mask Medium Mask Mouthpiece Large Mask

© 2013 Monaghan Medical Corporation™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. This device was Made in the USA from US and Canadian parts.

Manufactured by:

monaghan™

Monaghan Medical Corporation, 5 Latour Ave., Suite 1600, Plattsburgh, NY 12901
www.monaghanmed.com

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