AeroChamber® aVHC* Mask TIPS

It's important to remember the following tips to gain a child's cooperation:

-  Never force the mask on the child or attempt to treat while the child is crying.
-  Give praise and rewards whenever the child is using the chamber correctly.
-  Practice 'short' pretend treatments by having the mask near the child's face.
-  Use teddy bears, dolls or even adults to 'pretend' to use the chamber and make it look like a game.
-  Let the child play with the chamber while offering encouragement.
-  Give the treatment when the child is happy, playing or sleeping.
-  Give the child stickers to put on their chamber to make it their own.

Small Mask

Medium Mask

Small Mask

Medium Mask

This device has not been made (or manufactured) with BPA (Bisphenol A) or Latex.

Manufactured by:

monaghan.

Monaghan Medical Corporation, 5 Latour Ave., Suite 1600, Plattsburgh, NY 12901
www.monaghanmed.com

Remember to read the complete instructions that are packaged with your **AeroChamber®** anti-static Valved Holding Chamber (aVHC) device.

*aVHC = Anti-Static Valved Holding Chamber
© 2012 Monaghan Medical Corporation™ and ® are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. This device was Made in the USA from Canadian and US parts.

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